

# Condoms Intense



The natural rubber latex used for our quality condoms is produced in a fair trade manner. Our manufacturer receives a fair trade premium, which is used exclusively to improve the living and working conditions of the plantation workers and their families. [www.fairrubber.org](http://www.fairrubber.org)



- MEDICAL PRODUCT
- FAIR RUBBER CERTIFICATE FOR RAW MATERIALS SOURCED FROM FAIR TRADE, VEGAN
- FREE FROM MICROPLASTICS, BPA, PVC, THIURAM AND PHTHALATES.
- 100 % FAIR TRADE NATURAL RUBBER
- STUDDED, TRANSPARENT, MOISTURISED, CYLINDRICAL SHAPE, WITH RESERVOIR
- NOMINAL WIDTH: APPROX 53 MM, WALL THICKNESS: APPROX 0.085 MM
- A CO-OPERATION WITH GERMAN AIDSHILFE. LANGUAGES ON THE PACKAGING: DE

#### IMPORTANT INFORMATION BEFORE USING A CONDOM

This product is made of natural rubber latex. People who are allergic to natural rubber should not use this product. Please store in a dry place. Keep away from sunlight. Do not use twice. Medical device class 2b. Please read the instructions for use before use. 1. If extra lubrication is needed, use water-based lubricants. 2. DO NOT USE OIL BASED LUBRICANTS such as petroleum jelly, mineral oil and cold cream, as they may weaken the condom and cause subsequent failure of the product. So can some topical medicines that are applied to the penis or vagina. If you are not sure about the effect of the medicines you're using on condoms, ask your doctor or pharmacist. 3. Do not use the condom, if the rubber material is sticky or brittle. If the individual container is obviously damaged, discard that condom and use a new one from undamaged package. 4. Seek medical assistance as soon as possible, at least within 72 hours, should a condom leak or burst during use. 5. Condoms are for single use only, reused condom could increase the risk of breakage and infection. 6. When used correctly every time you have sex, latex condoms help prevent pregnancy, reduce the risk of transmitting HIV/AIDS and other sexually transmitted infections. 7. No method of contraception can provide 100% protection against pregnancy or transmission of HIV and STIs. If you suspect that you or your partner may have become pregnant or infected, consult your doctor immediately. 8. This product contains natural rubber latex, which may cause allergic reactions including anaphylactic shock. If you are allergic to natural rubber latex products, consult your doctor for advice before use. 9. Store in a cool dry place away from direct sunlight. 10. Keep away from children. 11. There is a risk of choking using condoms orally. 12. The use of additional lubricants during anal sex can reduce the risk of breakage. 13. Condom is a medical device to be used by adult males or females. 14. Report any serious incident occurred from using the device to the manufacturer and the relevant competent authority. 15. These condoms conform to International standards ISO 4074/ EN ISO 4074.

#### INSTRUCTIONS FOR THE USE OF A CONDOM

(1) Carefully tear off foil pack, making sure not to damage the condom. (2) During removal from foil, avoid damage to the condom by fingernails, jewelry and etc. (3) Squeeze the reservoir end of the condom gently, between the thumb and the forefinger to expel the air out of the tip, while placing the condom over the head of the erect penis. This must be done before any contact occurs between the penis and the partner's body to assist in the prevention of sexually transmitted infections and pregnancy. (4) While still holding the reservoir tip, unroll the condom over the entire length of the erect penis with the other hand. (5) Stop and check, if the condom is slipping off or tightening excessively onto the penis as this might lead to breakage. (6) After ejaculation, hold on the rim of the condom and slowly withdraw the penis, while it is still erect, so that the condom does not slip off. (7) Remove the condom from your penis by sliding it off slowly. It is important not to permit the penis or used condom to touch the vagina area after withdrawal. (8) Wrap the used condom and dispose it hygienically in a closed rubbish bin.

PRODUCT NAME	ART. NO.	UNIT	EAN CODE
FAIR SQUARED CONDOMS INTENSE, 10 PCS.	4910605	6	4260663812556
FAIR SQUARED CONDOMS INTENSE, 50 PCS.	4910607	6	4260663810996

• TARIFF NO.: 40141000



FAIR SQUARED GmbH  
Hermann-Heinrich-Gossen-Straße 4  
D-50858 Köln (Cologne), Germany  
Phone +49 (0) 2234 929 66 00  
[info@fairsquared.com](mailto:info@fairsquared.com)  
[www.fairsquared.com](http://www.fairsquared.com)